

Flying with your **PAP Device**

If you use a PAP device (CPAP, APAP, Bilevel) and need to travel for work or pleasure, you may wonder if there are any rules regarding traveling with and utilizing your PAP device. React Health is here to help you navigate flying with your PAP Device.

Know Before You Go...

1 Empty your humidifier chamber before getting ready to travel with your Luna PAP Device. We recommend that you do not pack your PAP device in your checked luggage. Luggage can sometimes be delayed even when you have a direct flight or when your flight is on time. Carrying your Luna PAP device in its carry case or in a carry-on will ensure that you have everything you need to sleep well when you get to your destination. Please remember to carry with you any needed documentation for your PAP device including, but not limited to:

- Your Prescription
- Letter certifying that your PAP device is compliant with FAA Regulations.
 - » For React Health Luna Devices, you can find this letter [HERE](#)

(continued on back)



Luna G3 CPAP
Luna G3 APAP



Luna G3 Bilevel 25A
Luna G3 Bilevel ST 30VT



Luna TravelPAP

(continued from front)

Your PAP Device and TSA Security (US)...

2

The TSA.gov website provides this guidance for carrying your PAP device through security checkpoints in the United States

“A CPAP, BiPAP and APAP must be removed from it’s carrying case and undergo X-ray screening. Facemasks and tubing may remain in the case. You may provide a clear plastic bag to place the device through the X-ray.” (TSA.gov)

Airlines and Baggage Policies...

3

Most U.S.-Based Airlines do not count a PAP device in it’s carrying case as part of your carry-on baggage allowance. We recommend checking with your specific airline prior to flying to confirm individual airline’s policies regarding carry-on baggage.

Using your PAP Device onboard the Aircraft...

4

If your airline allows the use of your compliant PAP onboard the aircraft:

- » You should not rely on a working power port for your PAP device in

your seat

Specific US Airlines Policies...

5

For your convenience below are some links to common U.S.-based Airlines policies regarding PAP device at the time of publication of this pamphlet. Please note that airline policies may change at any time and you should check with your airline prior to your flight for the most up to date resources.

- » [American Airlines](#)
- » [Delta Airlines](#)
- » [Jet Blue](#)
- » [United Airlines](#)

(Note: 48 hours advance notice required)

If you do not see your airline listed, please contact them directly.

Travel outside of the United States...

6

It is recommended that you contact your airline directly to discuss individual airline requirements. If you are traveling outside of the United States, please be aware that the power requirements may differ. The Luna

devices can be used with any standard electrical converter, but you will need to make sure you are bringing the proper adapter for the specific country you are traveling to. For patients using devices that have a humidifier, please be aware that tap water quality can vary greatly globally. React Health recommends the use of distilled water per the patient user manual for humidification.



**This is meant to be an informational guide only. React Health always recommends all patients check with their individual airlines to determine if PAP devices can be used on aircraft.*

...Enjoy Your Trip