


# The LightTrip (US) App

The LightTrip (US) App is a complimentary app that serves as a valuable enhancement to PAP therapy.

## What can the LightTrip (US) App Do?

- 1 Show helpful videos
- 2 Track your compliance to therapy
- 3 Set comfort features
- 4 Provide reminders for resupply
- 5 Conduct Mask Fit

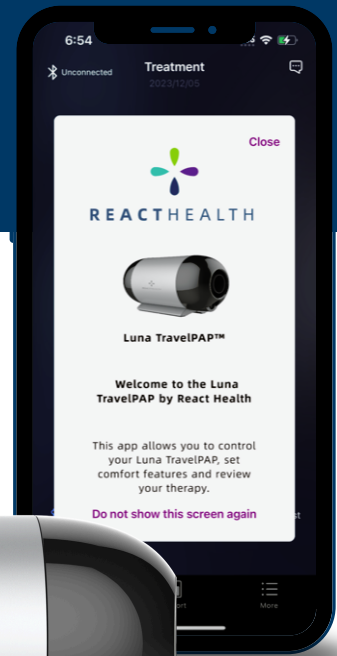

## Connecting Your Device

Have the device turned on and the app open. The Bluetooth light on your device will start to flash.

**When you open the app it will search for your serial number. Once you see your serial number, click on it and the device will connect.**

\*First time users of the app must watch the 3 instructional videos and then acknowledge the Terms & Conditions to enter the App.

## Prompt Messages

Your TravelPAP will periodically send you “Prompt Messages” that will appear in your LightTrip (US) App. To view your Prompt Messages, click on the message icon in the top right of the Treatment page.

Click the message and enter the message details page to view the message.

**Download in the App Store/Google Play**




**Apple App Store**



**Google Play**



REACTHEALTH

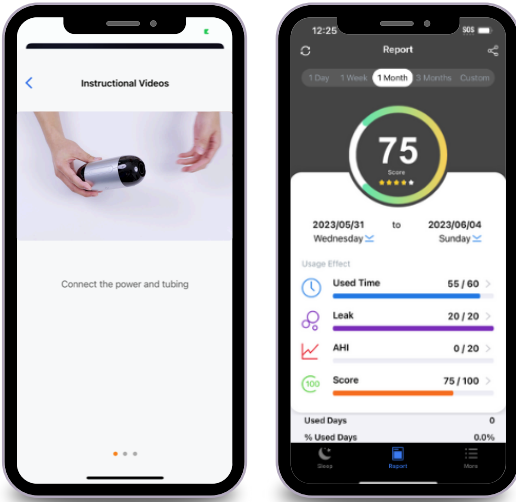

## 1 Show Helpful Videos

There are several instructional videos available in your app.

In order to access the instructional videos, go to the “More” tab on bottom navigation bar on the app homepage and click “Instructional Videos.”

Select the mask you will be wearing with the device to access the videos.

## 2 Track your compliance to therapy

To track compliance for your therapy, click on the Report icon on the bottom navigation bar to enter the “Reports” page.

When the device is connected to the LightTrip (US) App, this page will automatically send you a pop-up alert if there is new data available.

You can view reports for any time frame with the “Custom” button or choose from a preset range from 1 day to 3 months.

## 3 Set comfort features

There are several options in the Patient Menu for setting comfort features

- ✓ Ramp Time
- ✓ Reslex
- ✓ Auto On/Off
- ✓ Tubing Type
- ✓ Mask Type

To choose or change any setting, go into the “Settings” tab, set your feature at your preferred setting and click “Save.”

## 4 Provide reminders for resupply

You have the ability to set resupply reminders for your mask, tubing and filters within the app

When the device is connected to the App, click the Accessories icon to enter the Accessories page

Click “Set” to select the reminder cycle for each accessory

You can reset any of the reminder options by selecting the accessory and choosing “Reset”

## 5 Conduct Mask Fit

Click on the Fitting Test icon to enter the test page.

Put your mask on and click “Start” to begin the Mask Fitting Test. If the screen displays “Please adjust”, adjust the fit of your mask.

If the screen displays “Great!”, your mask is fitted properly.