

Luna TravelPAP™ Clinical Menu Settings

1. Steps to Navigate the Clinical Menu

The Clinical Menu is intended for use by clinicians only.

A. Access the Main Interface

Search and download the LightTrip (US) App from the App store on your mobile phone.

Connect the device to a power supply, press the **Bluetooth®** button on the main device, and the Bluetooth® indicator light will begin flashing. Open the LightTrip (US) App, click the Bluetooth® icon in the top left corner of the App and search for the device with the matching serial number. (The device serial number can be found on the device nameplate) If the device is connected to the LightTrip (US) App by Bluetooth® wireless technology, the Bluetooth® indicator light on the main device will turn blue.

After the connection via Bluetooth® wireless technology between the device and the LightTrip (US) App is completed, the software displays the Main Interface shown in Fig. 1-1.

Note: The Clinical Menu can only be entered and set when the device and the LightTrip (US) App are connected by Bluetooth® wireless technology.

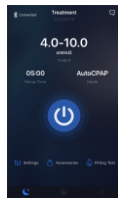




Fig.1-1

B. Enter the Initial Setup Interface

From the Main Interface shown in Fig.1-1, press and hold the **Start/Stop Button** and **Settings button**  +  for five seconds. The screen displays the Initial Setup Interface of the Clinical Menu, as shown in Fig.1-2.

The screen displays therapy, comfort and maintenance settings. Scroll down to access each menu.



Fig.1-2

C. Adjusting the Clinical Settings

Take the **Min APAP** as an example. Click to select the setting to be modified, adjust to the desired setting, then click "OK", as shown in Fig.1-3.

The clinical, comfort and maintenance settings can all be adjusted by following this process.



Fig.1-3

D. Exit the Clinical Settings Menu

After adjusting all applicable settings, scroll down to the bottom of the screen and click "Save". You will receive a prompt to verify the changes, click "OK" to confirm. To exit the settings menu select the back < icon at the top of the Settings Menu to go to the Main Interface shown in Fig 1-1

/

2. Description of Therapy Menu Settings

Mode		Range	Description
CPAP	Auto CPAP		
Initial P	Initial P	4.0 ~ 20.0 cmH ₂ O	As you adjust the setting, the number increases or decreases by 0.5 cmH ₂ O.
Treat P	—	4.0 ~ 20.0 cmH ₂ O	This pressure is the fixed treatment pressure after ramp time. You can adjust the setting by 0.5 cmH ₂ O.
—	Min APAP	4.0 ~ 20.0 cmH ₂ O	As you adjust the setting, the number increases or decreases by 0.5 cm H ₂ O.
—	Max APAP	4.0 ~ 20.0 cmH ₂ O	As you adjust the setting, the number increases or decreases by 0.5 cmH ₂ O.
SmartC	—	On / Off	In CPAP mode, you can set the SmartC on or off. When the SmartC is set to On, the device will adjust the treatment pressure automatically based on the sleep events of the patient over the past few nights.
—	SmartA	On / Off	In AutoCPAP mode, you can set the SmartA on or off. When the SmartA is set to On, the device will adjust the Initial P, Min APAP and Max APAP based on the sleep events of the patient over the past few nights.
—	Sensitivity	Standard, Soft, Female	This function allows you to set the sensitivity of the device to determine the forcefulness of the auto pressure adjustment needed depending on the events.
Leak Alert	Leak Alert	On / Off	If the patient's mask accidentally falls off during use, the device status light will flash. If the device is in the wireless connection state, it will send a prompt message via Bluetooth® wireless technology to the LightTrip (US) App. Prompting the patient to reapply the mask. This feature is available only when Auto Off is turned off.

3. Description of Comfort Menu Settings

Options	Range	Description
Ramp Time	Auto / 0 ~ 60 minutes	The ramp time can be set from 0 to 60 minutes or Auto. You can increase or decrease ramp time in 5 minute increments.
Reslex	Off / 1 ~ 3	This feature enables the device to automatically detect the patient's respiratory rhythm and reduce pressure during expiration, to make the patient more comfortable.
Auto On	On / Off	This feature enables the device to start automatically and deliver air at a preset pressure after the patient has taken a few deep breaths with the mask on.
Auto Off	On / Off	This feature enables the device to automatically discontinue the therapy and shut down when the mask is removed.
Tubing Type	15 mm/22 mm	There are two types of tubing available. Click to select "15 mm" or "22 mm".
Mask Type	Full Face Mask / Nasal Mask / Nasal Pillows Mask / Other Mask	There are three mask types available, Full Face Mask, Nasal Mask and Nasal Pillows Mask. But the patient can choose other masks as well. When the patient selects a masks other than one of the three types of REACT HEALTH masks, the device will identify the mask as other.

4. Description of Maintenance Menu Settings

Options	Range	Description
Pressure calibration	—	This feature enables the device to perform 0 cmH ₂ O and 20 cmH ₂ O pressure calibration. This option is only to be used by an Authorized Service Technician.
Mask test	Start	Measures mask leakage, and classifies non-REACT HEALTH masks as Other.
Erase Data	—	Choose to erase all data in the APP or erase all data stored on the device.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by REACT HEALTH is under license. Other trademarks and trade names are those of their respective owners.

V1.0
391567
Issue date: October 13, 2023